kusch|co

kuschmed®
HYGIENIC-LINE

cleaning and disinfection

CLEANING AND DISINFECTION

Cleaning and disinfection.
Reinigen und Desinfizieren.
Reinigen en ontsmetten.
Nettoyage et désinfection.

kuschmed®
HYGIENIC-LINE

English	4
Deutsch	12
Nederlands	20
Français	28

How to clean and disinfect kuschmed[®] surfaces correctly

Cleaning and disinfection instructions for the surfaces of our products. Please carefully read and follow all the instructions given in this manual. These instructions help you to preserve the hygienic and germicidal properties. For the best possible long-term hygienic safety!


Only use uncoloured disinfectants approved by the DGHM, the German institute for hygiene and microbiology and the VAH, the German Association of Applied Hygiene. Always strictly observe the manufacturer's instructions and respect the recommended surface contact time of the dilute solution.

Always try out the solution on an invisible spot before applying it to the surface. Having no influence whatsoever on the production or

the composition of the substances listed in this manual, we can not accept liability for any damages resulting from the use of these substances.

We advise against changing the brand of disinfectant – especially when you have been using a particular brand for a long time, as the brands might be incompatible.

Disinfectants are no cleaning agents! Please observe the separate cleaning instructions.

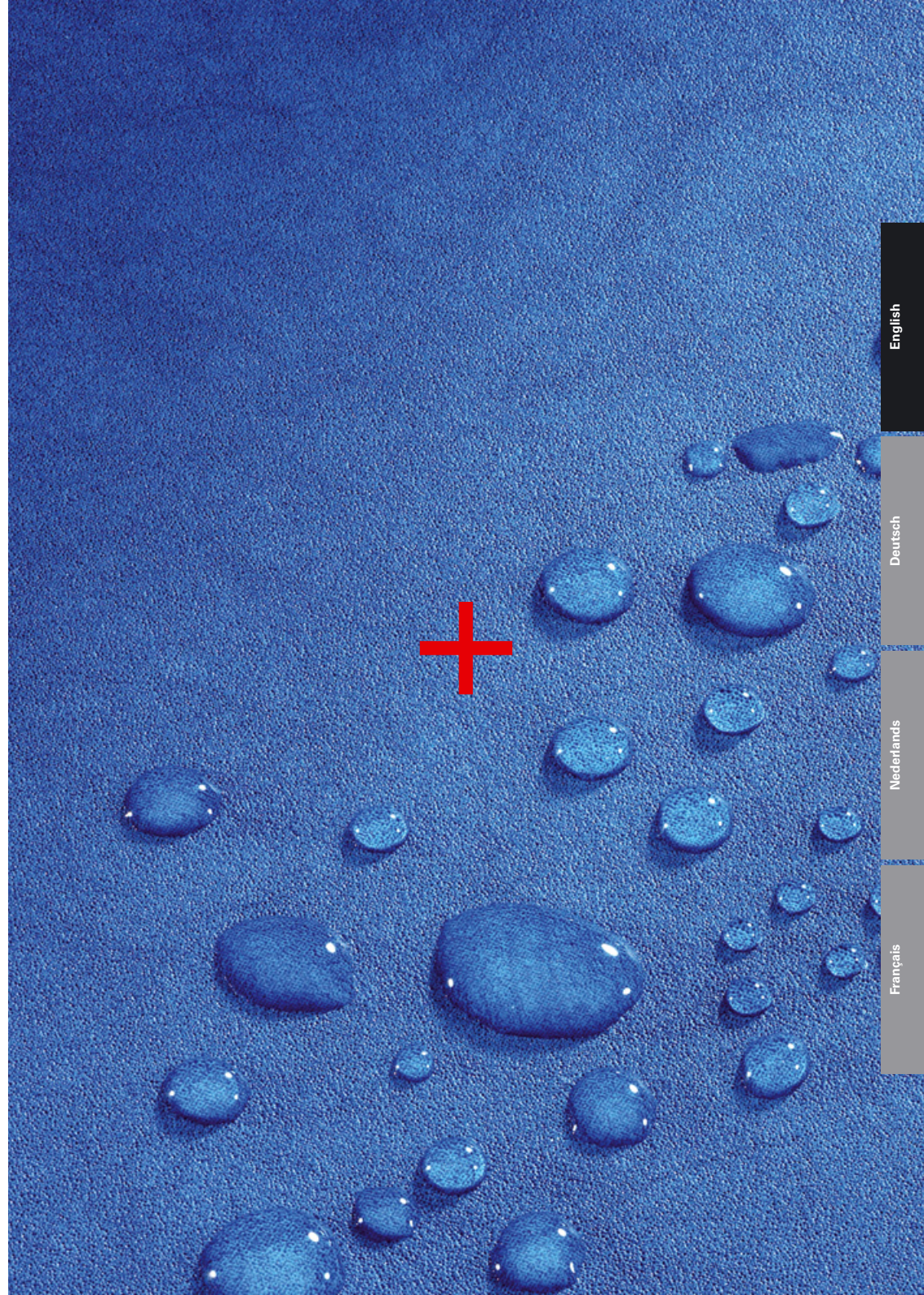**Fabric
Skai Palena**

Cleaning instructions
Wipe the surface clean with a damp cloth. We recommend the use of a microfibre cloth. Stubborn stains can be removed with a soft brush and some mild detergent. Wipe spilled liquids immediately off, before they dry in.

The following disinfectants are approved:
e.g. Terralin 0.5 %,
Perform 3.0 %,
Indicin Plus 0.5 %,
Indicin Extra N 0.5 %,
Indicin Foam gf*,
Mikrobac forte 0.5 %,
Meliseptol gf*,
Kohrsolin 1 %,
Hexaquart S 0.5 %,
etc.

Attention

Never apply benzine to the fabric.
Do not wash or dry-clean.

**Fabric
Virtual Plus**

Cleaning instructions
Wipe the surface clean with a damp cloth. We recommend the use of a microfibre cloth. Stubborn stains can be removed with a soft brush and some mild detergent. Wipe spilled liquids immediately off, before they dry in.

The following disinfectants are approved:
Terralin Liquid gf*,
Mikrozid AF wipes,
Antifect FD 10 2 %,
Perform 1 %,
Terralin Protect 0.5 %,
Bacillol AF gf*,
Dismozon pur 1 %,
Kohrsolin extra 2 %,
Mikrobac forte 0.5 %,
Hexaquart plus 0.5 %,
Meliseptol gf*,
Indicin Foam gf*,
Indicin Rapid 2 %,
Indicin Plus 0.5 %

Attention

Never apply benzine to the fabric.
Do not wash or dry-clean.

* gf = ready-to-use

**Fabric
Nova Care and
Spectrum Care**

Cleaning instructions
The fabric can be vacuumed or wiped clean with a soft brush and some hand-hot water. Damp stubborn stains with some soapy water (ratio water-mild detergent 10 : 1) and let it take effect. Then, brush from the edge towards the middle and dab afterwards with a dry cloth.

The following disinfectants are approved:
Terralin Liquid gf*,
Mikrozid AF wipes,
Antifect FD 10 2 %,
Perform 1 %,
Terralin Protect 0.5 %,
Bacillol AF gf*,
Dismozon pur 1 %,
Kohrsolin extra 2 %,
Mikrobac forte 0.5 %,
Hexaquart plus 0.5 %,

Meliseptol gf*,
Indicin Foam gf*,
Indicin Rapid 2 %,
Indicin Plus 0.5 %

Attention

Do not wash or dry-clean.

**Fabric
Niroxx**

Cleaning instructions
Wipe the surface gently clean with a damp sponge or a damp sponge cloth. Never use a dry cloth. Dab the dampened spot afterwards. Normal stains can be wiped clean with a damp sponge and some soapy water. Fresh water/oil-based stains can be removed with Niro-cleaner. Please observe the manufacturer's cleaning instructions. Stubborn stains can be removed with isopropyl alcohol.

The following disinfectants are approved:
Quartamon Med,
Mikrozid Liquid,
Kodan wipes,
Mikrozid AF wipes,
Terralin Liquid gf*,
Perform 1 %,
Terralin Protect 0.5 %,
Hexaquart plus 0.5 %

Attention

Never use stain removers or solvents.
Do not wash or dry-clean.

Metal surfaces

- **kuschmed® antibacterial powder coating**
- **satin or bright chrome-plating**

Cleaning instructions

These surfaces are low-maintenance and are easy to clean with standard cleaning agents, for instance, dilute solutions containing low concentrations of washing-up liquids, soft soap or window cleaner. Only use soft cloths or sponges for cleaning.

When the powder coating film is damaged, it might set about the corrosion process. This process might lead to the formation of rust and the coating might start to chip off. That is why wet surfaces should be dried immediately to avoid standing water.

kuschmed® antibacterial wood lacquer finish

Cleaning instructions

Dust/normal stains:

Use a damp, non-fluffy cloth (either cotton or linen or alternatively a chamois) and add a few drops of washing-up liquid. Wipe the surface clean with the damp cloth (attention: the cloth may not be wet) without applying pressure. Wipe the surface clean with a dry cloth. If needed, you may add other cleaning agents, such as all-purpose cleaners like Dor or Sidolin and washing-up liquids like Pril, Palmolive, Fairy Ultra etc.

Stubborn stains:

Please follow these steps: For stubborn stains, you may increase the dosage of the washing-up liquid or the mild cleaning agents, however never in concentrated form! Alternatively, you may use alcohol-based cleaning agents in the recommended dosage. Always wipe the surface with a damp cloth (never wet!). Always rub the surface dry to avoid standing water in the pores! Never use scouring powder or any other abrasive cleaner, such as ATA, Viss, etc. as the grains in the abrasive damages the lacquer finish. We also strongly advise against aggressive bathroom and toilet cleaners.

The following disinfectants are approved:

Kohrsolin FF 3 %,
Perform 1 %
Indicin Plus 1 %,
Dismozon pur 1 %,
Mikrobac forte 3 %,
Ultrasol F 1 %

Attention

Never rub or scour the surface. Do not use aggressive solvents or abrasive cleaners. Avoid alkaline or acid cleaning agents.

The following disinfectants are approved:

Descocid 2 %,
Ultrasol F 1 %
Biguamed Perfekt N 0.5 %
Terralin Liquid gf*,
Mikrozid AF wipes,
Antifect FD 10 2 %,
Perform 1 %,
Terralin Protect 0.5 %,
Bacillol AF gf*,
Dismozon pur 1 %,
Kohrsolin extra 2 %,
Mikrobac forte 0.5 %,
Hexaquart plus 0.5 %,

Meliseptol gf*,
Indicin Foam gf*,
Indicin Rapid 2 %,
Indicin Plus 0.5 %

Attention

Do not use aggressive solvents or scouring agents. Avoid alkaline or acid cleaning agents.

RESOPAL®+HIPERCARE®

Cleaning instructions

Lightly soiled tops can be wiped with a soft, damp cloth. Remove sticky stains with some warm soapy water or standard cleaning agents, free from any scouring agents as abrasive cleaners scratch the surface finish. Tenacious stains, such as varnish, lacquer,

glue, ink, lipstick, etc. can generally be removed with organic solvents, such as acetone, trichloroethylene, alcohol, etc.

The following disinfectants are approved:

Ultrasol F 1 %
Perform 1 %
Biguamed Perfekt 0.5 %
Meliseptol gf*
Bacillol AF gf*,
Descocid 2 %

Attention

Never use abrasive cleaners!

Hard plastic components

Cleaning instructions

Regularly clean the plastic surface to remove stains and dust. You use either a soft brush or a soft sponge; alternatively you may use a micro-fibre cloth. The sponge can be dampened with some standard plastic cleaner or some hand-hot water and a mild cleaning agent.

Never use solvent-containing cleaning agents without having tested them on an invisible spot. Use turpentine or benzene on stains only.

Soft plastic components (integral foam)

Cleaning instructions

Regularly clean the plastic surface to remove stains and dust. You use either a soft brush or a soft sponge; alternatively you may use a micro-fibre cloth. The sponge can be dampened with some standard plastic cleaner or some hand-hot water and a mild cleaning agent.

Never use solvent-containing cleaning agents without having tested them on an invisible spot. Use turpentine or benzene on stains only.

The following disinfectants are approved:

Meliseptol gf*,
Antifect FD 10 2 %,
Perform 1 %,
Terralin Protect 0.5 %,
Dismozon pur 1 %,
Kohrsolin extra 2 %,
Mikrobac forte 0.5 %,
Hexaquart plus 0.5 %,
Indicin Foam gf*,
Indicin Rapid 2 %,
Indicin Plus 0.5 %

Attention

Never use abrasive cleaners, as they might damage the surface structure.

The following disinfectants are approved:

Bacillol AF gf*,
Descocid 2 %,
Ultrasol F 1 %
Perform 1 %
Biguamed Perfekt N 0.5 %
Terralin Liquid gf*,
Mikrozid AF wipes,
Antifect FD 10 2 %,
Terralin Protect 0.5 %,
Dismozon pur 1 %,
Kohrsolin extra 2 %,
Mikrobac forte 0.5 %,

Hexaquart plus 0.5 %,
Meliseptol gf*,
Indicin Foam gf*,
Indicin Rapid 2 %,
Indicin Plus 0.5 %

Attention

Never use abrasive cleaners, as they might damage the surface structure.

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GmbH & Co KG